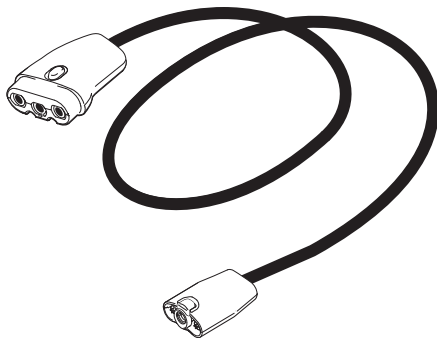
# 14' MEDICAL CONNECTOR HOSE

14' Medical Connector Hose (PN 570324)

### Intended Purpose

The Game Ready™ Connector Hoses are intended to be used together with the Game Ready™ Control Unit (GRPro™ 2.1) and Game Ready™ Wraps as a system to treat post-surgical and acute injuries in order to reduce edema/swelling and pain for which cold and compression therapy are indicated. The system is intended to be used by or on the order of licensed healthcare professionals in hospitals, rehabilitation facilities, outpatient clinics, athletic training settings, or home settings. Clinical benefits include reduction of post-surgical- and acute injury-associated pain and edema/swelling.

### Warnings

- Do not use any harsh or abrasive materials on the Medical Hose
- Do not use any phenolic based cleaners on the hose
- Do not subject the hose to any sterilization protocols (i.e. autoclave or ethylene oxide sterilization)

Warning: To comply with California Proposition 65, the following warning has been included: This product contains chemicals known to the State of California to cause cancer, birth defects or other reproductive harm.

**Notification to Users/Patients:** Please immediately report any serious incident that may have occurred due to this medical device to the manufacturer or its Authorized Representative and to your National Authority.

### Cleaning Instructions

The surface of your Game Ready™ Medical Connector Hose may be wiped down as needed using soap and water.

To decontaminate the surface, certain commercially available agents may be used, always following the instructions provided with the agent. We recommend Quaternary Ammonia based cleaning agents, such as Cavicide®. When possible, we recommend the use of Caviwipes® or equivalent. A liquid form may also be applied using a soft cloth. In all cases, follow the cleaning agent manufacturer's instructions to determine the necessary exposure time for proper disinfection.

### Maintaining Hose Connection Points

Your hose connection points may require lubrication from time to time. Use only silicone based lubricants as petroleum based products will destroy the components of the Connector Hose and void your warranty. You can obtain silicone spray from Game Ready by contacting Customer Service at 1.888.GameReady (1.888.426.3732).

### Warranty

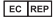
The Game Ready™ Medical Connector Hose has a warranty of one year from the date of purchase.

Contact Game Ready Customer Service with questions or for further information at 1.888.GameReady (1.888.426.3732).

 Medical Device	 Rx Only			 Importer	 Unique device identifier
--	---	---	---	--	--

Distributed in the USA by CoolSystems, Inc., 1800 Sutter St., Ste. 500, Concord, CA 95420 USA.  
In USA, 1-888-GAMEREADY (1-888-426-3732). [www.gameready.com](http://www.gameready.com)

 CoolSystems, Inc.,  
DBA Game Ready,  
1800 Sutter St. Ste. 500,  
Concord, CA 94520 USA.

 Emurgo Europe,  
Prinsessegracht 20,  
2514 AP The Hague,  
The Netherlands.

 Avanos Medical Belgium BVBA,  
Leonardo da Vinciiaan 1,  
1930 Zaventem, Belgium.


Sponsored in Australia by Emurgo Australia  
201 Sussex Street, Darling Park, Tower II, Level 20  
Sydney NSW 2000, Australia.

\* Registered Trademark or Trademark of CoolSystems, Inc., or its affiliates.  
© 2020 CoolSystems, Inc. All rights reserved. 2021-04-30 15-M1-957-04 / 50000468

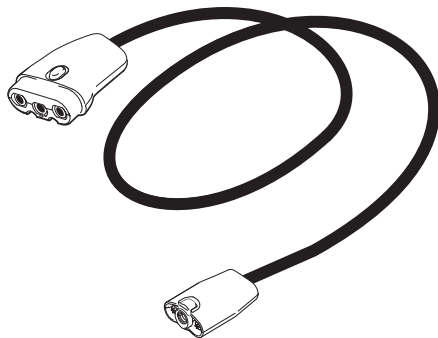Cavicide is a Registered Trademark of Microecology Products Corp.  
Caviwipes is a Registered Trademark of Metrex Research LLC.

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© 2020 CoolSystems, Inc. All rights reserved. 2021 年 6 月 7 日 15-M1-1482-01