# DUAL CONNECTOR HOSE

## Intended Purpose

The Game Ready® Connector Hoses are intended to be used together with the Game Ready® Control Unit (GRPro™ 2.1) and Game Ready® Wraps as a system to treat post-surgical and acute injuries in order to reduce edema/swelling and pain for which cold and compression therapy are indicated. The system is intended to be used by or on the order of licensed healthcare professionals in hospitals, rehabilitation facilities, outpatient clinics, athletic training settings, or home settings. Clinical benefits include reduction of post-surgical- and acute injury-associated pain and edema/swelling.

**Notification to Users/Patients:** Please immediately report any serious incident that may have occurred due to this medical device to the manufacturer or its Authorized Representative and to your National Authority.

## Getting Started

- Connect the single end (the side with the red connector button) of the Dual Connector Hose to your Game Ready® Control Unit.
- Apply both Wraps to the patient(s) being treated. Make sure the Wraps are applied snugly and correctly.
- Connect the ends of the Dual Hose to the Wraps you are using for treatment†.
- Verify that the Dual Connector Hose is securely connected to both Wraps and to the Control Unit.
- Press the Control Unit's Start Button. The Control Unit will provide the same level of cold and compression to both Wraps.

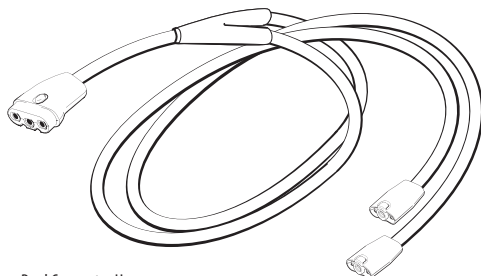
Note: Cold and compression in each Wrap cannot be controlled individually.

## Usage Tips



There are a few things that you can do to maximize the performance of your Dual Connector Hose:

- Try and minimize any difference in elevation between the two Wraps being used.
- Make sure that both Wraps are applied snugly to the body.
- Verify that the Dual Connector Hose is attached securely to both Wraps and is not kinked.
- Do not remove either Wrap during treatment. Always pause the Control Unit before removing a Wrap.
- Never use the Dual Connector Hose with a single Wrap; the Dual Connector Hose will only provide treatment to two Wraps simultaneously.

Note: When different body parts are treated simultaneously on the same patient, the patient may feel that one Wrap is providing more cold or compression than another even if both Wraps are functioning properly. This is a result of differing sensitivities of different body parts to both cold and compression.

Dual Connector Hose  
(PN 570302)

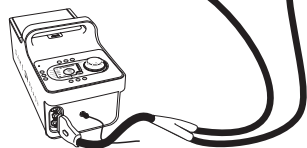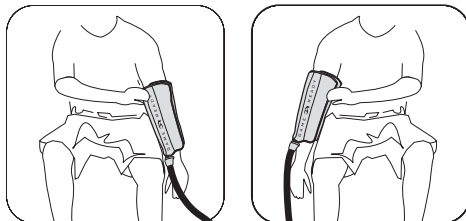† The Dual Connector Hose must be attached to two Wraps. It cannot be used with only a single Wrap.

## Care and Storage

The Dual Connector Hose can be cleaned using soap or mild detergent and water. Using any other cleaning or sterilizing agent may damage or destroy the hose and void all warranties. The Dual Connector Hose should be stored coiled. It must not be left kinked or compressed.

## Warranty

The Game Ready® Dual Connector Hose has a warranty of one year from the date of purchase.

Contact Game Ready Customer Service with questions or for further information at 1.888.GameReady (1.888.426.3732).

Warning: To comply with California Proposition 65, the following warning has been included: This product contains chemicals known to the State of California to cause cancer, birth defects or other reproductive harm.

 Medical Device	Rx Only			 Importer	 Unique device identifier
--	---------	---	---	--	--

Distributed in the USA by CoolSystems, Inc., 1800 Sutter St., Ste. 500, Concord, CA 95420 USA.  
In USA, 1-888-GAMEREADY (1-888-426-3732). [www.gameready.com](http://www.gameready.com)

 CoolSystems, Inc.,  
DBA Game Ready,  
1800 Sutter St. Ste. 500,  
Concord, CA 94520 USA.

  Emergo Europe,  
Prinsessegracht 20,  
2514 AP The Hague,  
The Netherlands.

 Avanos Medical Belgium BVBA,  
Leonardo da Vinciiaan 1,  
1930 Zaventem, Belgium.

Sponsored in Australia by Emergo Australia  
201 Sussex Street, Darling Park, Tower II, Level 20  
Sydney NSW 2000, Australia.

\* Registered Trademark or Trademark of CoolSystems, Inc., or its affiliates.

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© 2020 CoolSystems, Inc. All rights reserved. 2021年6月10日