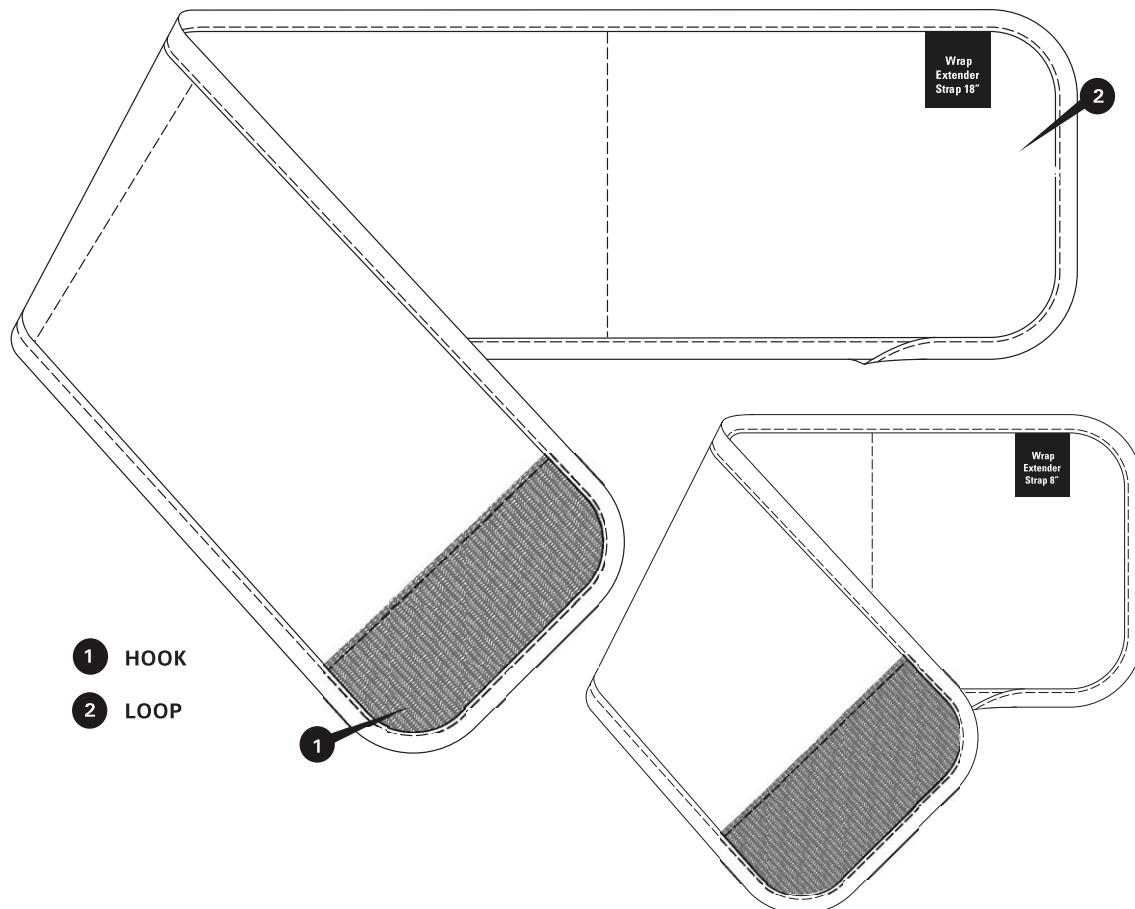
# WRAP EXTENDER STRAPS – 18" & 8"

ACCESSORY FOR USE WITH  
**HIP/GROIN WRAP**  
**SHOULDER WRAP**

## USE GUIDE

ENGLISH

The Wrap Extender Straps can be used with the Game Ready\* Shoulder and Hip/Groin Wraps as directed in these instructions.

The Wrap Extender Straps should only cover areas of the joint or body that do not receive therapy. The Wrap Extender Straps do not extend the cooling area of the wrap.

### CARE AND CLEANING

Hand wash in lukewarm water with mild detergent. Line dry only.

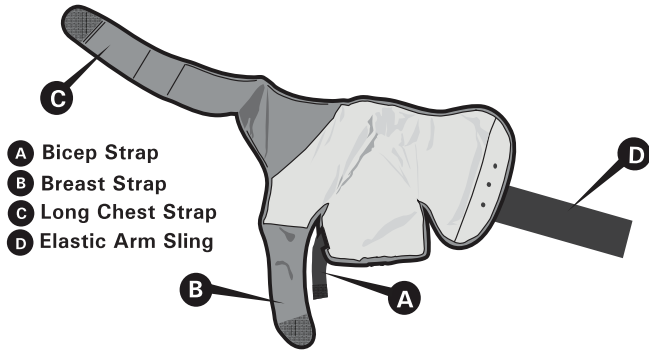
### INTENDED PURPOSE

The Game Ready\* Wraps are intended to be used together with the Game Ready\* Control Units (GRPro\* 2.1 or Med4 Elite\*) as a system to treat post-surgical and acute injuries in order to reduce edema/swelling and pain for which localized thermal therapy (hot or cold or contrast) are indicated. The system is intended to be used by or on the order of licensed healthcare professionals in hospitals, rehabilitation facilities, outpatient clinics, athletic training settings, or home settings. Clinical benefits include reduction of post-surgical- and acute injury-associated pain and edema/swelling.

**ASSEMBLY & APPLICATION**  
FOR USE WITH GAME READY® SHOULDER WRAP



Note: To extend the Shoulder Wrap, apply the 8" or 18" Wrap Extender Straps PRIOR to placing the Shoulder Wrap on the body.

- Place the Shoulder Wrap on a flat surface, blue side up. Ensure all the straps from the wrap are unhooked and are laid flat.

- To extend the bicep strap (A), use the 8" Wrap Extender Strap:

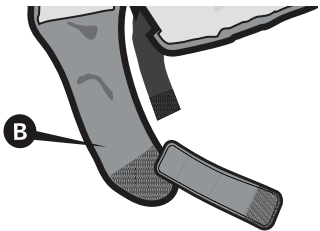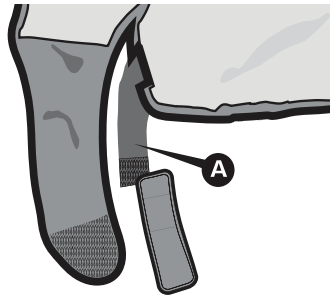
- Hold the Wrap Extender Strap with the hook-side facing up and towards you.
- Secure the bicep strap hook to the underside (loop-only side) of the Wrap Extender Strap.

- To extend the breast strap (B), use the 8" Wrap Extender Strap:

- Hold the Wrap Extender Strap with the hook-side facing up and towards you.
- Secure the breast strap hook to the underside (loop-only side) of the wrap extender strap.

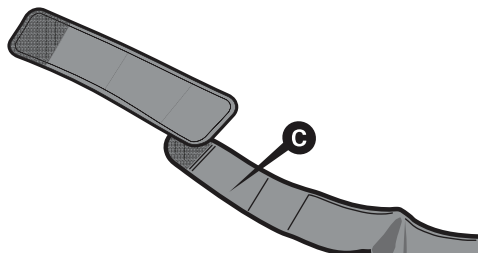- To extend the long chest strap (C), use the 18" Wrap Extender Strap:
  - Position the Shoulder Wrap with the long chest strap positioned towards you.
  - Hold the Wrap Extender Strap with the hook side facing up and towards you.
  - Secure the long chest strap hook to the underside (loop-only) of the Wrap Extender Strap.


- Apply the Shoulder Wrap to the body, per the Shoulder Wrap Use Guide.
- Once the Shoulder Wrap is on the body, adjust to a comfortable fit.


**ASSEMBLY & APPLICATION**  
FOR USE WITH GAME READY® HIP/GROIN WRAP

Note: To extend the Hip/Groin Wrap, apply the 8" or 18" Wrap Extender Straps PRIOR to placing the Hip/Groin Wrap on the body.

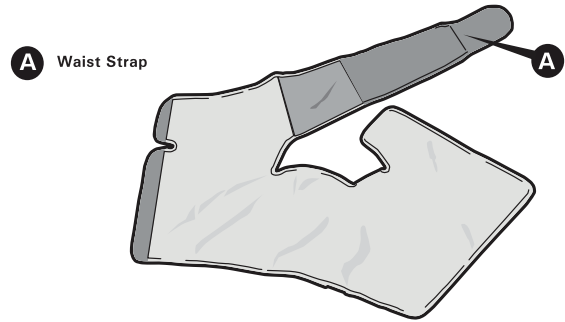- Place the Hip/Groin Wrap on a flat surface, blue side up. Ensure the waist strap is unhooked and is laid flat.

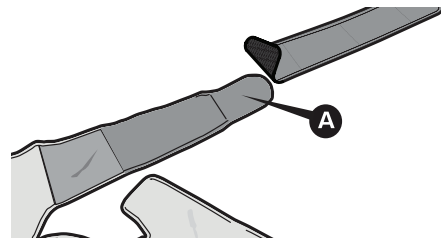- Hold the Wrap Extender Strap hook-side down and facing away from you.
- Secure the hook end of the Wrap Extender Strap onto the loop of the waist strap (A).

- Apply the Hip/Groin Wrap to the body, per the Hip/Groin Wrap Use Guide.
- Once the Hip/Groin Wrap is on the body, adjust to a comfortable fit.

**CONTACT US**

In the U.S. call Game Ready Customer Service at 1.888.426.3732 (+1.510.868.2100). From outside of the U.S. please refer to [www.gameready.com](http://www.gameready.com) to find your local distributor's contact information.

Symbol for "assembled in" a specific country (XXXX).

Symbol for "made in" a specific country (XXXX).

Medical Device

Unique Device Identifier

Importer

**Warning:** To comply with California Proposition 65, the following warning has been included: This product contains chemicals known to the State of California to cause cancer, birth defects or other reproductive harm.

**Notification to Users/Patients:** Please immediately report any serious incident that may have occurred due to this medical device to the manufacturer or its Authorized Representative and to your National Authority.

Last Revised: 2021-04-29  
YYYY-MM-DD

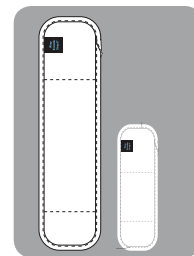
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

COOLSYSTEMS, INC.  
DBA Game Ready  
1800 Sutter Street, Suite 500  
Concord, CA 94520 USA  
1.888.GAMEREADY +1.510.868.2100  
[www.gameready.com](http://www.gameready.com)

\* Registered Trademark or Trademark of CoolSystems, Inc., or its affiliates. ©2020 CoolSystems, Inc. All rights reserved. Game Ready® Wrap Extender Straps Use Guide

EMERGO AUSTRALIA  
201 Sussex Street  
Darling Park, Tower II, Level 20  
Sydney NSW 2000  
Australia

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